

INSTRUCTIONS FOR Heavy Duty Metal Pill Crusher (#7355)



Step 1. Place the ½ oz. Paper Souffle Cup (#7368) into the Pill Crusher.



Step 2. Place the tablet into the cup.



Step 3. Cover the tablet with a second cup.



Step 4. Use an up and down tapping motion to crush the tablet. Excessive force is not necessary.



Step 5. Remove the cups from the crusher and administer the medication. No cleanup required; just dispose of the cups.

Don't crush enteric coated tablets.

This crusher can be used with ½ oz. paper souffle cups.