

# INSTRUCTIONS FOR Micro Cart (#20493)

**Step 1.** Mount all 4 casters to the base.

**Step 2.** Mount legs A and B to the front side of the base. Note: Choose one of the long sides as front.

**Step 3.** Mount legs C and D to the back of the base.

**Step 4.** Mount the middle shelf with the open lip to the front.

**Step 5.** Mount the top shelf with the open lip to the front.

**Note:** Apply a drop of Loctite to the 8 screw threads that mount the legs to the base. This will help keep the screw from working its way loose.

