



INSTRUCTIONS FOR HCL® LUD Press 2.0 - Easy Fill Vials (#20932)



Step 1. Remove the ¼" socket head cap screw from the back of the unit.

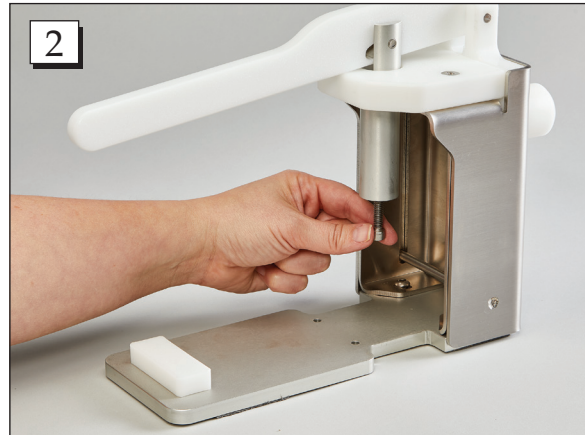

Step 2. Thread the cap screw into the bottom of the anvil as far as it will go.

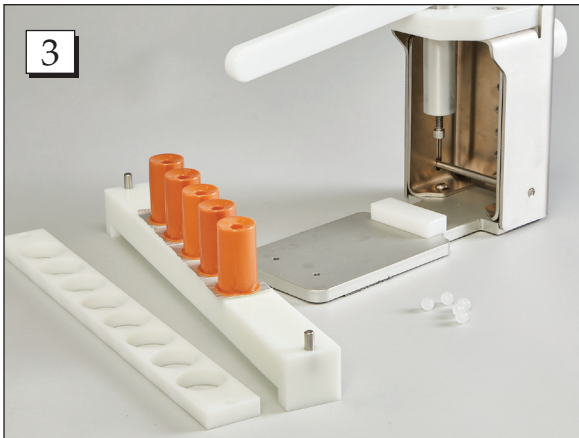

Step 3. With the top of the carrier removed, place the Easy Fill Vials onto the base.



Step 4. Place the top of the carrier over the Easy Fill Vials and onto the locating pins.



Step 5. Fill the vial with medication. Then place the carrier in between the guide block and the frame.



Step 6. Place the Easy Fill ball over the sealing hole in the vial and gently press the ball into place.

INSTRUCTIONS FOR HCL® LUD Press 2.0 - Tamper Tuf Vials (#20932)



Step 1. Remove the anvil insert from the back of the unit.



Step 2. Thread the anvil insert into the bottom of the anvil.



Step 3. For Tamper Tuf Vial "Unassembled," place the Vial with Cap into the carrier. Then place the carrier in between the guide block and the frame, under the press. It should move freely to the left and right. For Tamper Tuf Vial "Assembled," skip to step 5.



Step 4. Press the cap onto the vial with firm pressure.



Step 5. Fill the vial with medication using the port hole in the top of the cap. Flip the port seal cover over the opening in the filled vial and firmly press to close.

Call Free: 1.800.848.1633 • Fax Free: 1.800.447.2923
GoHCL.com • hcl@GoHCL.com

INSTRUCTIONS FOR HCL® LUD Press 2.0 - Bottles (#20932)



Step 1. Remove the thumb screw from the back of the unit.



Step 2. Slide the top upward. Choose the appropriate hole for the back thumb screw based on the bottle height. Use the lowest hole available. Then insert and tighten the thumb screw.



Step 3. Fill the bottle with medication. Place the Adapter Cap into the bottle opening.



Step 4. Gently press the Adapter Cap into the bottle.