

CLEANING GUIDE FOR Lock-To-Pole IV Lock Box (#19169 & 19170)

Wipe surfaces with 70% Isopropyl Alcohol, taking care to avoid electronic components. Allow to air dry per dwell time provided on disinfectant label.

Warnings:

Disinfectant should be compatible with materials listed below. Some disinfectants are not compatible with polycarbonate. Discontinue use if signs of damage occur.

Materials:

Polycarbonate
POM (acetal)
TPE (Thermoplastic Elastomer)
Polypropylene

Alternate Disinfectant Option:

19954, PREempt™ One-Step Surface Cleaner and Disinfectant Wipes, Canister
19954-31, PREempt™ One-Step Surface Cleaner and Disinfectant Wipes, Case