


INSTRUCTIONS FOR


Lock-To-Pole IV Lock Boxes (#18634, 18638, 18855 & 18875)



Step 1. Inspect the clamps for signs of wear and tear. Open the door of the IV box and make sure the 2 clamps are open. Note: The box will not properly attach to the pole if the clamps are not in the open position.

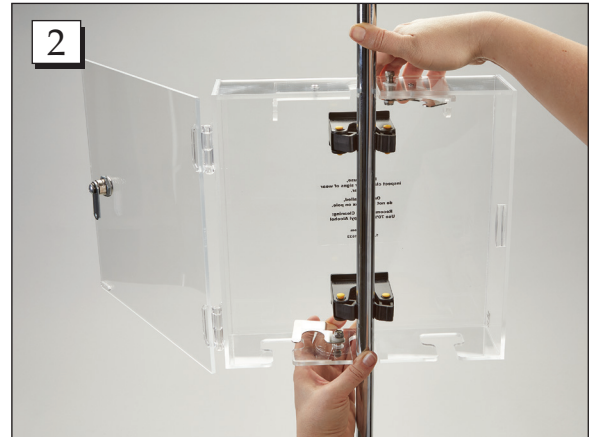

Step 2. Align the clamps on the inside of the box with your pole. Holding the box with your four fingers and the pole with your thumb, press firmly until the clamps wrap around the pole. Once installed, do not twist the box on the IV pole.

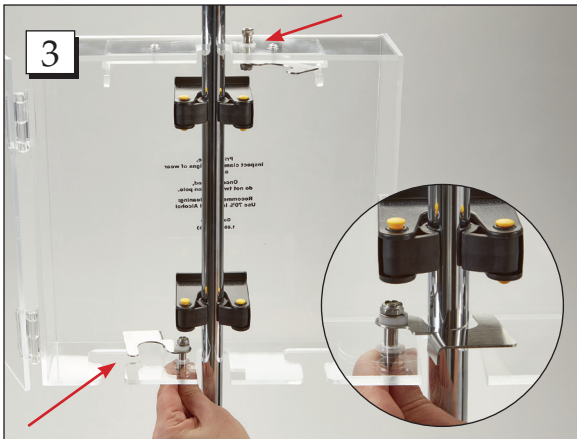

Step 3. Press and twist the safety latches located at the top and bottom of the box to the locked position.



Step 4. Hang the IV bags on the hooks.



Step 5. Lock the IV box door.

Health Care Logistics will not be liable if product is used outside of scope of the instructions.

Call Free: 1.800.848.1633 • Fax Free: 1.800.447.2923
GoHCL.com • hcl@GoHCL.com

© Health Care Logistics, Inc. 2025

CLEANING GUIDE FOR

Lock-To-Pole IV Lock Boxes (#18634, 18638, 18855 & 18875)

Warnings:

Disinfectant should be compatible with materials listed below. Some disinfectants are not compatible with acrylic. Discontinue use if signs of damage occur.

Materials:

Acrylic
POM (acetal)
TPE (Thermoplastic Elastomer)
Polypropylene

Recommended Disinfectant Options:

19954, PREempt™ One-Step Surface Cleaner and Disinfectant Wipes, Canister
19954-31, PREempt™ One-Step Surface Cleaner and Disinfectant Wipes, Case

Alternate Disinfectant Options:

3% Hydrogen Peroxide Based Cleaners
15% Sodium Hypochlorite Based Cleaners