

ASSEMBLY INSTRUCTIONS FOR


Drug Storage Wall Desk (#7448)

NOTE: Be sure to select the appropriate mounting hardware to ensure safe and reliable use. Mount hardware through a stud, blocking or rigid structure where possible during installation.


Step 1. Position the WallWrite on wall or door in desired location and using the pre-punched holes as a template mark location of mounting holes (Keyhole slots). Make sure WallWrite is level.

Step 2. If hanging to wall studs. Drill 1/8" holes into the two marks through the wallboard and into the supporting studs. Insert two #8 x 1" screws into the two holes. Do not tighten all the way. Mount the WallWrite over these two screws and tighten.


Step 2a. If mounting to drywall or plaster wall use the KapToggle™ anchors supplied in hardware kit. Drill two 1/2" holes in wall where previously marked. Use below instructions to properly install anchors.

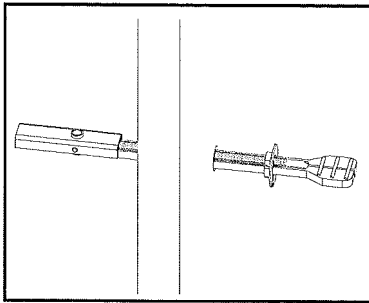


Push KapToggle™ thru drilled

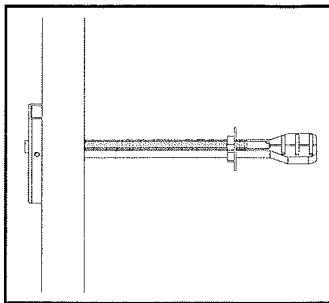

Pull KapToggle™ snugly against inside wall.

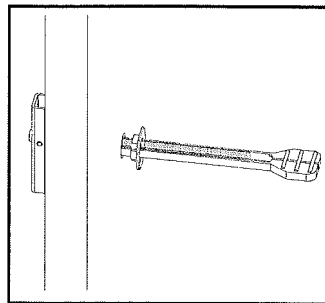

Slide washer evenly along straps until inner rim of washer seats completely inside drilled hole.

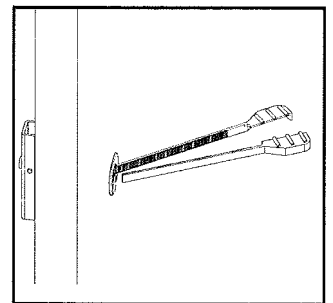


Snap off plastic straps flush with washer.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923

Web: www.GoHCL.com • Email: hcl@GoHCL.com

ASSEMBLY INSTRUCTIONS FOR

Drug Storage Wall Desk (#7448)

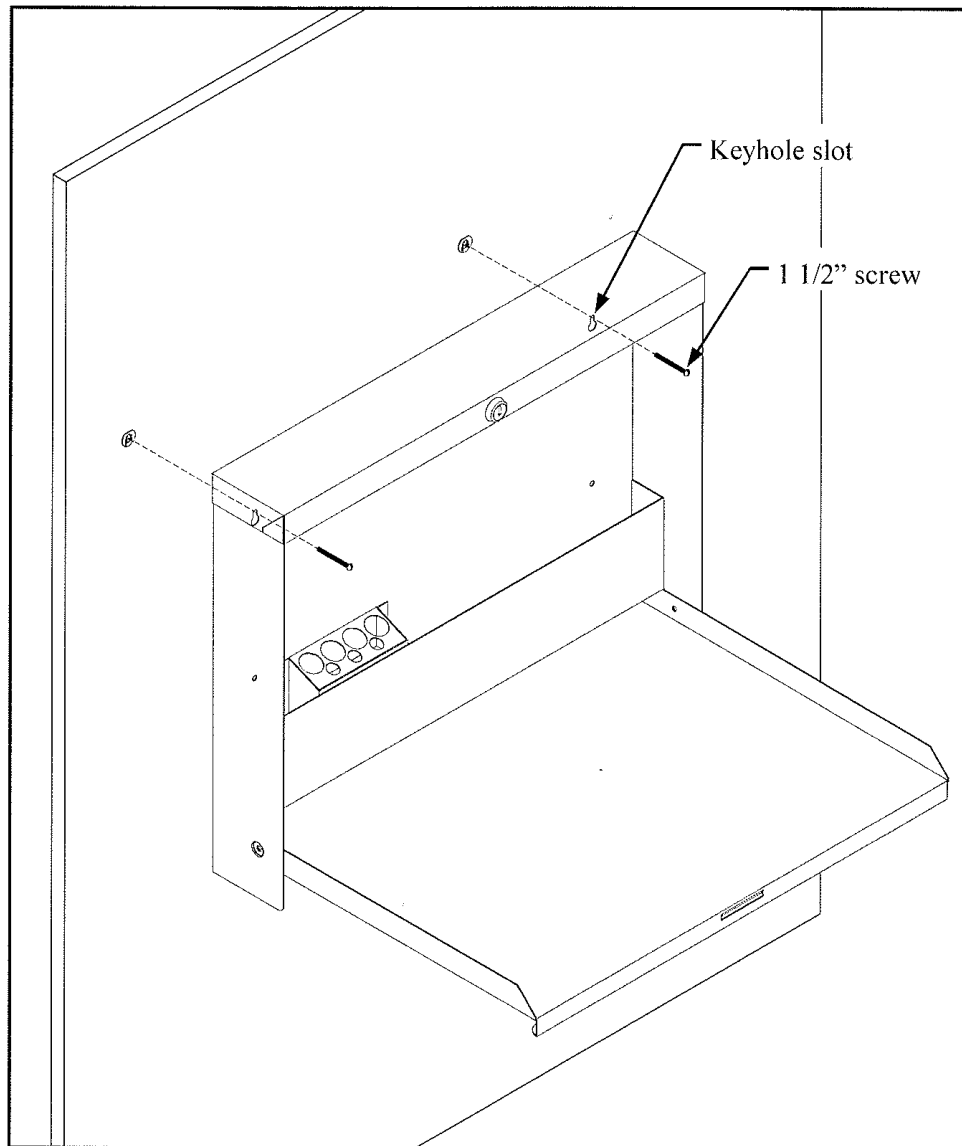

Insert 1 1/2" screw into KapToggle™ anchor. Do not tighten all the way. Mount WallWrite over these two screws and tighten.



Call Free 1.800.848.1633 • Fax Free 1.800.447.2923

Web: www.GoHCL.com • Email: hcl@GoHCL.com

ASSEMBLY INSTRUCTIONS FOR

Drug Storage Wall Desk (#7448)


Step 2b. If mounting to a concrete block wall use the plastic wall anchors. Drill two 3/16" holes. Insert anchors into the holes. Screw two #8 x 1" anchor screws into the two anchors. Do not tighten all the way. Mount the WallWrite over these two screws and tighten.

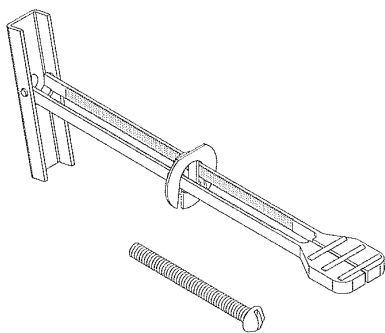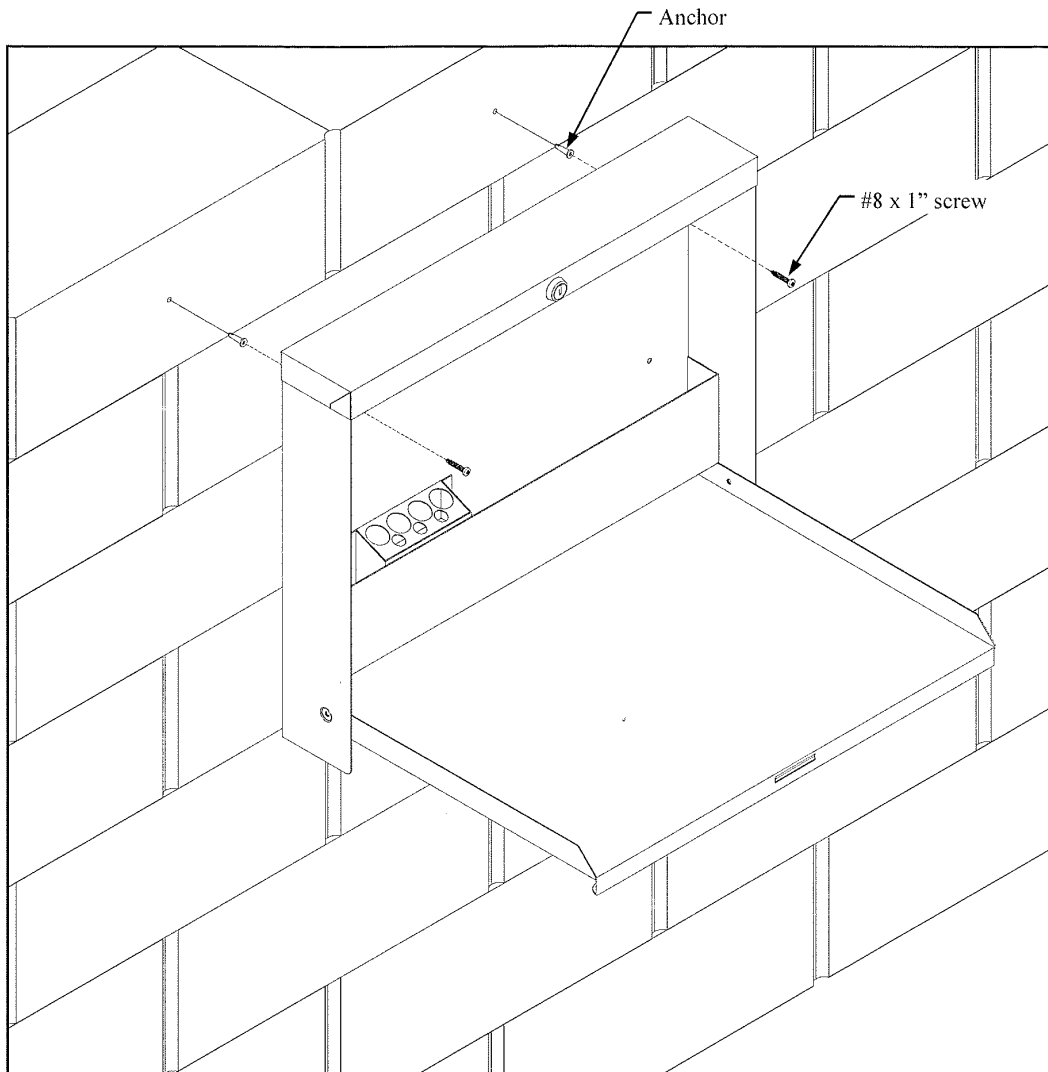

Use On: Drywall or plaster wall



Use On: Concrete block wall

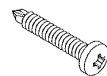

Use On: Wall studs or wood door

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923

Web: www.GoHCL.com • Email: hcl@GoHCL.com