


INSTRUCTIONS FOR HCL® Personal Tablet Crushers with Cap (#7335, 7338, 21294 & 21295)



Step 1. Unscrew and remove the crusher lid, then place the tablet into the base



Step 2. Replace the crusher lid and turn it clockwise until the tablet is crushed. Rotate the lid back and forth to achieve a finer crush, then dispense the medication or continue to **Step 3** to dilute it.

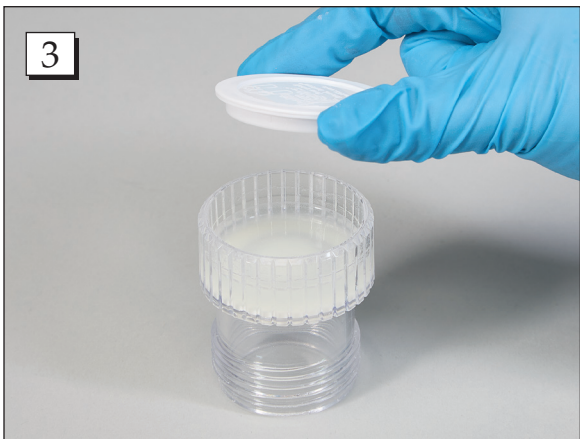

Step 3. Add water or juice to the crusher base, then snap the cap from the storage compartment onto the base.



Step 4. Shake the crusher to mix the medication with the water or juice.



Step 5. Drink directly from the crusher or draw the diluted medication into an oral syringe for administration.

Call Free: 1.800.848.1633 • Fax Free: 1.800.447.2923
GoHCL.com • hcl@GoHCL.com