

INSTRUCTIONS FOR INDIVIDUAL TABLET GRINDER/STORAGE (HCL #7332)

CAUTION:

Some medications such as time release or enteric coated tablets **SHOULD NOT** be crushed. Consult with a pharmacist or physician before proceeding. Thoroughly clean after each use.



1. Grasp the Tablet Grinder bottom in one hand and turn the top section counter clockwise with the other hand until the two sections are separated.




2. Insert tablet into the bottom section of the crusher.

3. Replace top portion and turn clockwise until two units are flush. Twist back and forth several times to grind tablet into powder.

4. Separate the two units by unscrewing, and tap out ground powder. Add to liquid or soft solid.

5. If desired the dishwasher-safe storage container may be used as a drinking vessel. Remove lid of top unit by flipping lid off with thumb.

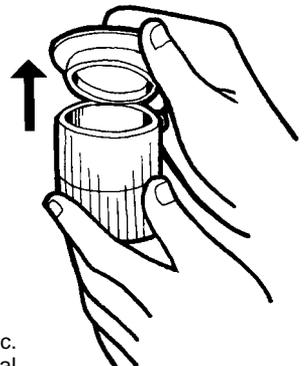

Distributed by Health Care Logistics, Inc.
Manufactured by AMI-American Medical Industries. Made in the USA

sheet #I7332

Rev. 1/18/95

Call Free 1-800-848-1633 • Fax Free 1-800-447-2923
International Call (740) 477-1686 • International Fax (740) 477-2923