

ASSEMBLY INSTRUCTIONS FOR

Utility Cart with 3 Tub Shelves (#17824)




PARTS LIST


- 1 - Bottom shelf w/caster sockets
- 1 - Top shelf w/handle
- 1 - Center shelf
- 8 - Legs
- 4 - Casters

Tools Required:


- 1 - Rubber mallet
- 1 - Slotted screwdriver

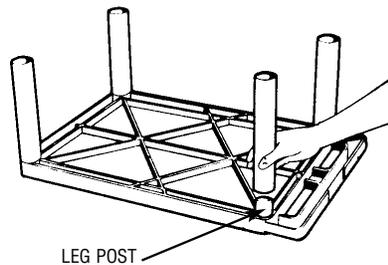


- 1.**
Lay top shelf (with handle), ribbed side up on a smooth surface. Push 4 legs onto leg posts.

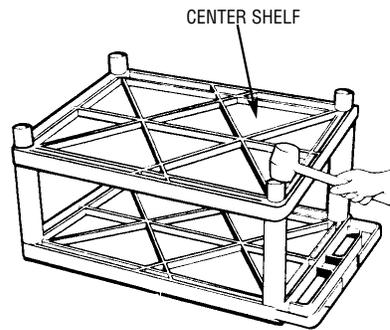


- 2.**
Position center shelf ribbed side up over the 4 legs and insert leg posts into legs. Using a rubber mallet, firmly hammer center shelf into legs until all four legs are seated (flush) against the surfaces of both shelves.

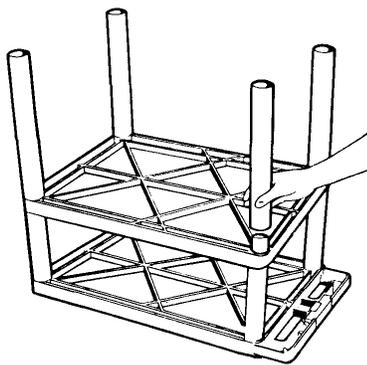

- 3.**
Push remaining legs onto leg posts of center tub shelf.

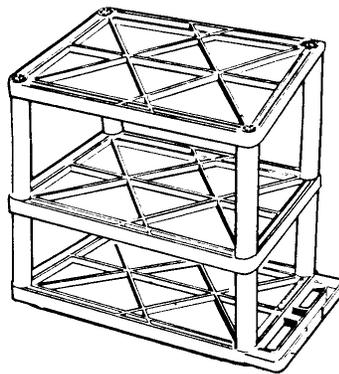

- 4.**
Position bottom shelf, ribbed side up, over the legs and insert leg posts into legs. Firmly hammer bottom shelf until seated onto all 4 legs.

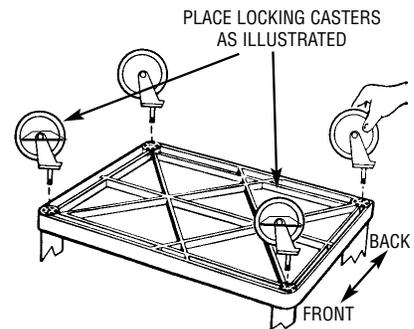

- 5.**
Push the four casters firmly into the holes in each corner of the bottom shelf. **Important:** casters must be fully seated. Tap firmly with rubber mallet to properly seat casters.

Call Free 1.800.848.1633 or 1.888.HCL-INTL Fax Free 1.800.447.2923
Website: www.HealthCareLogistics.com • Email: hcl@HealthCareLogistics.com