

CLEANING GUIDE FOR

HCL® LUD Press 2.0 (#20932)

Materials:

- 304 Stainless-Steel
- High-Density Polyethylene (HDPE)
- Anodized Aluminum

304 Stainless-Steel

Recommended Disinfectant Options:

20141 Sterile HCL® Presaturated Cleanroom Wipes, 70% IPA, 9 x 12
19118 Sterile Isopropyl Alcohol, 16 oz.

Alternate Disinfectant Options:

70% Isopropyl Alcohol Based Cleaners

High-Density Polyethylene (HDPE)

Recommended Disinfectant Options:

19954 PREempt™ One-Step Surface Cleaner & Disinfectant Wipes, 6 x 7, Canister
20141 Sterile HCL® Presaturated Cleanroom Wipes, 70% IPA, 9 x 12

Alternate Disinfectant Options:

70% Isopropyl Alcohol Based Cleaners

Anodized Aluminum

Recommended Disinfectant Options:

N/A

Alternate Disinfectant Options:

Mild detergent with a non-scratch cloth.

Follow the cleaner's instructions and adhere to the specific Standard Operating Procedure (SOP) established by your healthcare system.