

INSTRUCTIONS FOR


HCL® Personal Tablet Crushers with Cap (#7335, 7338, 21294 & 21295)



Step 1. Unscrew and remove the crusher lid, then place the tablet into the crusher base



Step 2. Replace the crusher lid and turn it clockwise until the tablet is crushed. Rotate the lid back and forth to achieve a finer crush, then dispense the medication or continue to **Step 3** to dilute it.

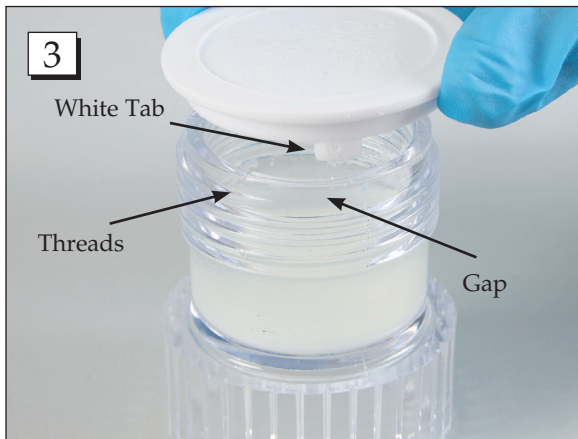

Step 3. Add water or juice to the crusher base, then snap the cap from the storage compartment onto the crushing reservoir. Make sure the white tab on the lid is positioned in the gap between the threads.



Step 4. Shake the crusher to mix the medication with the water or juice.



Step 5. Drink directly from the crusher base or draw the diluted medication into an oral syringe for administration.

Call Free: 1.800.848.1633 • Fax Free: 1.800.447.2923
GoHCL.com • hcl@GoHCL.com