RXCRUSH® PILL CRUSHING SYSTEM

RxCrush® is designed by a nurse for a nurse. A contained pill crushing system that delivers pulverized and dissolved medication to a feeding tube.

Directions

Follow these steps to crush, dissolve, and deliver medication to a feeding tube using the PC16 Pill Crusher II, PP06 Pill Pouch and PP08 Pill Pouch.

1

EN With gloved hands, open the plastic pouch and insert the medication pills.

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2

EN Attach the water-filled EnFit syringe to pouch. Do not inject the water yet.

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3

EN Position pouch zip seal under roller. Roll over the zip seal to ensure closure.

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4

EN Hold syringe to the side and use the crush plate to crush pills by moving handle up and down.

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5

EN Move the pill pouch under roller and roll back and forth to pulverize pill. Syringe should be held to the side again.

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6

EN Remove syringe and pouch from the crusher. Inject water from syringe into pill pouch.

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7

EN Keep syringe attached to pouch and shake pill pouch until medication is completely dissolved. Take the extra time to ensure it's completely mixed.

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8

EN Draw dissolved solution from pill pouch into syringe. Hold pouch up and syringe down to ensure contents flow into syringe.

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9

EN Hold syringe up and pouch down so contents do not leak. Detach syringe from pouch. Attach syringe to feeding tube and inject solution from syringe into feeding tube.

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10

EN Refill syringe with water, inject water into pouch, withdraw contents and inject into feeding tube. Repeat as many times as necessary to remove all pill particles from pill pouch.

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POUCH CADDY

Use attached Pouch Caddy to store pill pouches for an easier work flow.

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CLEANING

RxCrush® Pill Crusher is made of non-corrosive material and can be cleaned with water and disinfectant cleaners. Clean the RxCrush Pill Crusher using the RxCrush Cleaning Guide, available at RxCrush.com. Pill pouches are intended for single-patient use and are not intended to be cleaned or re-used.

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WARRANTY

The manufacturer warrants the RxCrush® Pill Crusher to be free of functional defects for five years. If the RxCrush Pill Crusher is used to crush anything other than pills in the RxCrush Pill Pouch, the warranty is null and void. The warranty is also void if damage is due to accident, misuse, or misapplication. To make a claim, contact us at www.RxCrush.com/contact.

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WE'RE HERE TO HELP

To reorder or for assistance, visit us at www.RxCrush.com or call +01.877.343.2783.

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