


Instructions for Disposable Needle Guard



1. Push the top portion of the needle guard out until it locks with the needle luer.

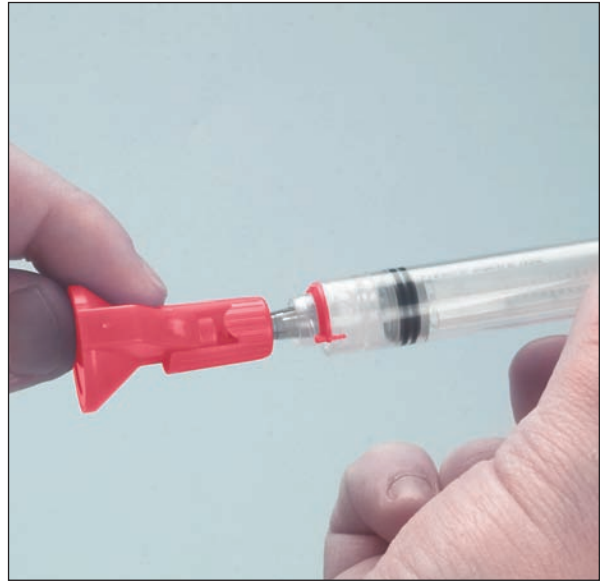

2. Turn the needle counterclockwise to remove it from the syringe.



3. The needle guard and needle can be placed in a disposal bag or sharps container.



HCL-INTL is a registered trademark of Health Care Logistics, Inc.

Sheet ##7220
REV.021508

Call Free: 1 800 848 1633 or 1 888 HCL INTL® (425 4685) Fax Free: 1 800 447 2923
Web Site: www.HealthCareLogistics.com • E-mail: hcl@HealthCareLogistics.com